



The Multi-T® - Multi Trainer


THE MULTI - T® is a pre-fixed appliance used during mixed dentition between the ages of 6 and 10 years old. It provides an eruption guide for both the incisors and the cuspids. The Multi-T® is useful for problems associated with swallowing, thumb sucking, tongue thrusting, and breathing.

Material: Silicone

Competition

Ortho-Tain Nite-Guide®

- Children ages 6-10 years old
- Eruption guide for both incisors and cuspids
- Reduce problems associated with: swallowing, thumb sucking, tongue thrusting, labial dysfunction, respiratory problems
- One size fits all

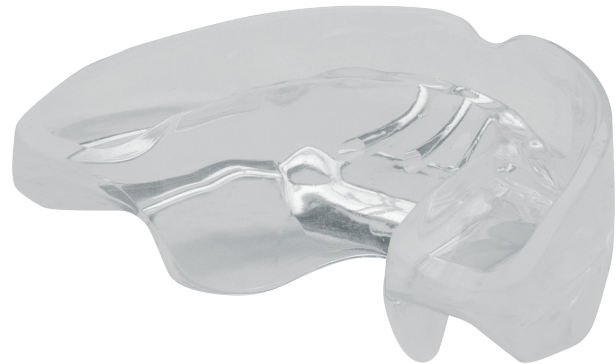

Type	Age	Sizes	Holes	Lip-Bumper Effect
Multi-T*	6-10	1	yes	yes

Type	Guidance
Multi-T*	Incisors and Canines

	Part Number	Package Contains
Multi-T*: Multi - Trainer Appliance	J00800	1 each
	J00801	5 each

Multi-Family™ Frequently Asked Questions

1. How do you clean it?

-Daily:

- o Before use rinse under running water
- o After use, wash the appliance with a toothbrush under running water (Toothpaste can be used occasionally if necessary. As toothpaste is mildly abrasive, excessive use of toothpaste may scratch the surface.)

-When needed:

- o Disinfect the appliance in boiling water
- o Wash the box in a dishwasher
- o Sterilization in autoclave at 273 F

2. How long does the patient have to wear it?

- As for fixed appliances, it is impossible to give an exact time for how long treatment will take. It is a very complex question because many factors come into play:

- o Type/severity of malocclusion
- o Skeletal growth pattern of patient
- o Patient specific individual characteristics of dental development
- o How early/late is treatment initiated
- o Clinical preferences of the clinician
- o Patient compliance

