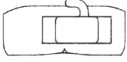
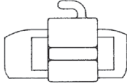
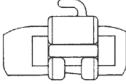
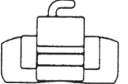


Maxillary Molar

Buccal

Placement
Mesial-Distal

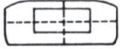

Occlusal-Gingival

			
Single Buccal Tube Mesial end of tube bisects mesial-buccal cusp	Convertible Double Tube Mesial end of tube bisects mesial-buccal cusp	Convertible Double Tube Mesial end of tube bisects mesial-buccal cusp	Convertible Double Tube Mesial end of tube bisects mesial-buccal cusp

Arch Slot for illustrations shown is 2.0mm ABOVE occlusal edge of band

Lingual

Placement
Mesial-Distal
Occlusal-Gingival


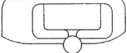
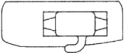

			
3D® Lingual Tube (Wilson®) Centered Occlusal edge of tube offset 1.0mm from occlusal edge of band	Lingual Arch Sheath Centered Centered	Lingual Button Centered Centered	Cleat Lug Centered Offset to Occlusal

Mandibular Molar

Buccal

Placement
Mesial-Distal


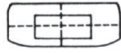


Occlusal-Gingival

			
Single Buccal Tube R & L Mesial end of tube bisects mesial-buccal cusp Arch Slot 2.0mm BELOW occlusal edge of band	Double Tube R & L Mesial end of tube bisects mesial-buccal cusp Arch Slot 2.0mm BELOW occlusal edge of band	Single Tube R & L Mesial end of tube bisects mesial-buccal cusp Arch Slot 2.0mm BELOW occlusal edge of band	Single Tube R & L Mesial end of tube bisects mesial-buccal cusp Tube even with gingival edge of band

Lingual

Placement
Mesial-Distal

Occlusal-Gingival

			
3D® Lingual Tube (Wilson®) Centered Occlusal edge of tube offset 1.0mm from occlusal edge of band	Lingual Arch Sheath Centered Centered	Lingual Button Centered on mesial cusp Offset to Occlusal	Lug/Cleat Centered Offset to Occlusal